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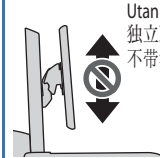
SV31: ≤ 10 lbs (4.5 kg)
SV32/SV32 LiFe/SV41/SV42: ≤ 15 lbs (6.8 kg)

SV31 Carts:

CAUTION: TIPPING HAZARD! If the combined LCD and CPU weight is greater than 20 lbs (9 kg) then the CPU must be mounted to the rear of the cart using the Universal CPU Holder accessory (ordered separately).

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SV32/SV41 Carts:

CAUTION: TIPPING HAZARD! If the combined LCD and CPU weight is greater than 23 lbs (10.4 kg) then the CPU must be mounted to the rear of the cart using the Universal CPU Holder accessory (ordered separately).

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SV42 SLA Carts:

CAUTION: TIPPING HAZARD! If the combined LCD and CPU weight is greater than 27 lbs (12.2 kg) then the CPU must be mounted to the rear of the cart using the Universal CPU Holder accessory (ordered separately).

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

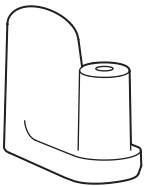
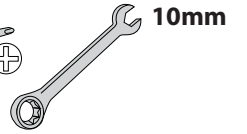
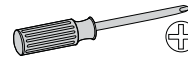
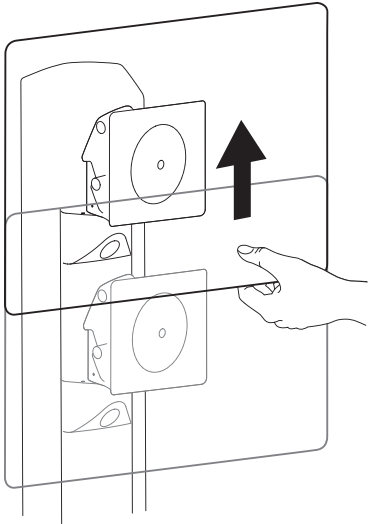
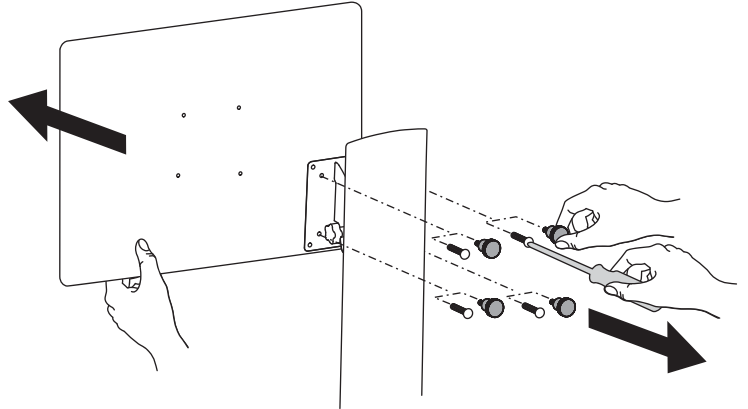
3

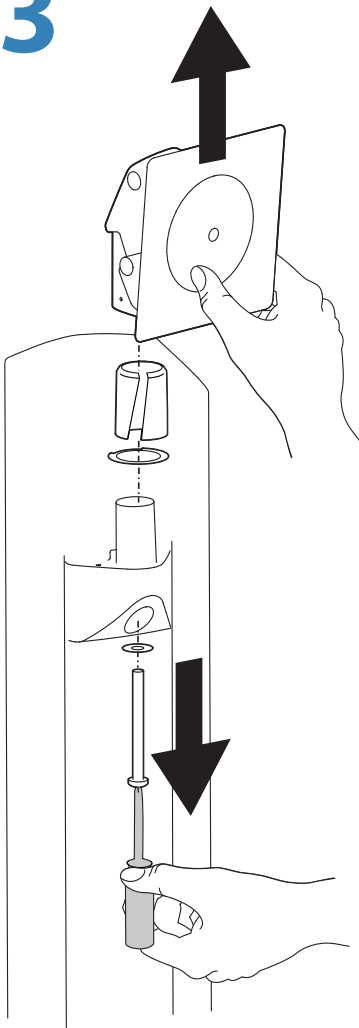
4

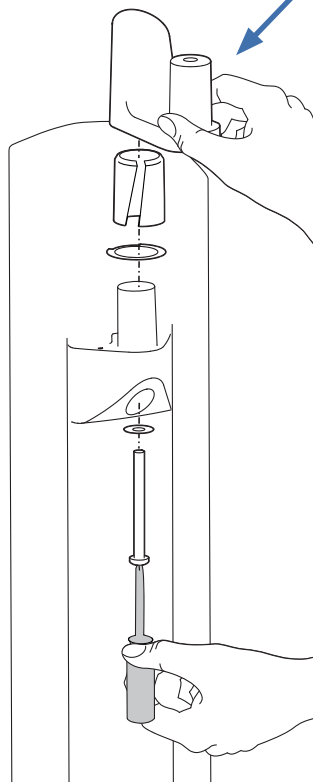
NOTE: Must seat fasteners fully to insure safety ground path.

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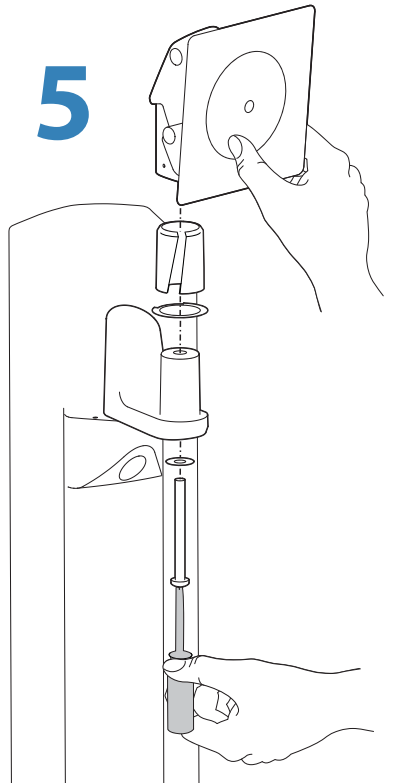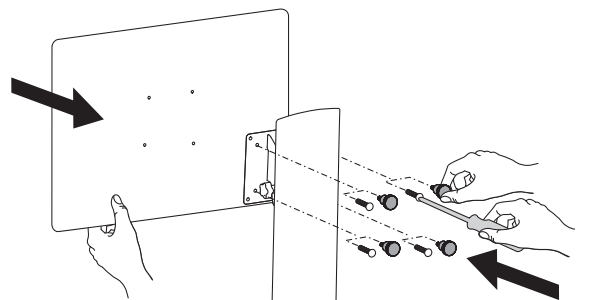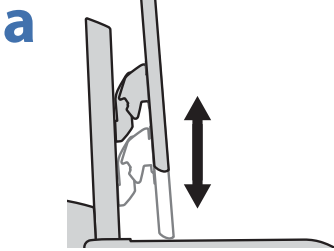
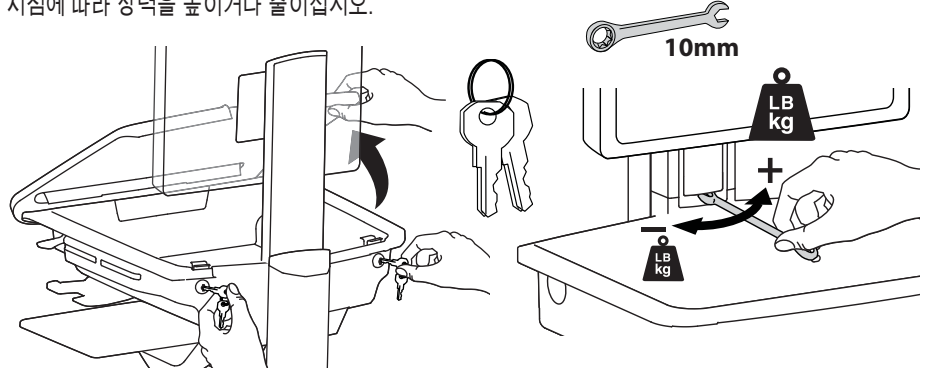
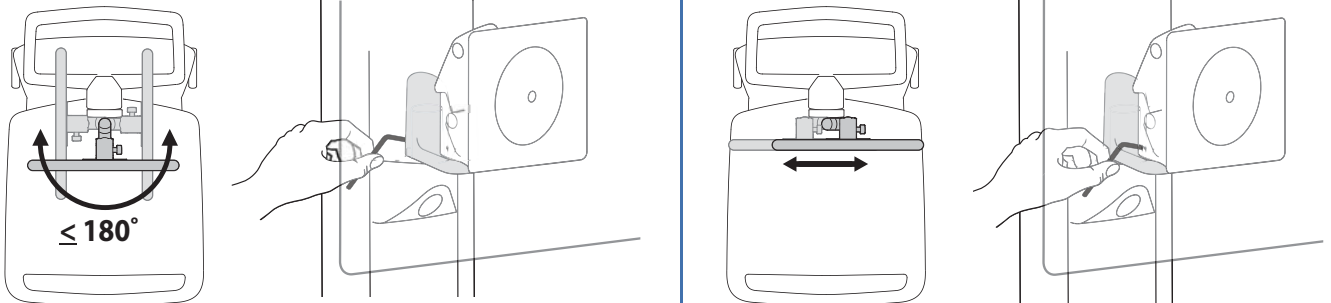
© 2012 Ergotron, Inc.
All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA. Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.

Americas Sales and Corporate Headquarters **EMEA Sales**
 St. Paul, MN USA Amersfoort, The Netherlands
 (800) 888-8458 +31 33 45 45 600
 +1-651-681-7600 www.ergotron.com
www.ergotron.com info.eu@ergotron.com
sales@ergotron.com

APAC Sales **Worldwide OEM Sales**
 Tokyo, Japan www.ergotron.com
www.ergotron.com info.oem@ergotron.com
apacustomerservice@ergotron.com